



CLINiC P1224T Quick Start Guide

Scan the QR code to open the User Guide >>>>

Unboxing the Unit

See unboxing document DOC-UPI-CLIN-A01 included in the box.

Unit Setup

Refer to the User Guide for complete instructions on setup, cleaning and use of the CLINiC. Use the following information to assist in assembly of your system components. You need a #1 and #2 Philips screwdriver.

1. Mount the CLINiC on a cart, stand or desk stand to prevent accidental damage during assembly
 - a. **DO NOT** lift the CLINiC by the monitor, as this can damage the screen. Support the unit from underneath and/or the chassis in the rear.
 - b. Standard VESA 100x100 mount from the rear of the unit. Select from upper position or lower position depending on preference. Use the M4 x 8mm screws provided. If longer screws are needed, verify they are not hitting objects inside the chassis.
2. Unbox the camera and locate the parts accessory box. Extra hardware may be provided.
 - Poly E60 Camera
 - Camera Mounting Bracket
 - (1) ¼-20 flathead screw for mounting the camera to the bracket
 - (1) M3 x 8MM flathead for camera bracket to camera

Use caution not to rest or hold the camera on the moving parts. **FRAGILE!**

 - a. Align bracket to the bottom of the camera as shown
 - b. Orient the bracket to the front of the camera as shown.
 - c. Install screws as shown.
3. Mount the Camera/bracket to the CLINiC.
 - (4) #6-32 x 3/8 flathead screws
4. Plug in the USB and Power cables to the back of the camera
5. Attach the Cable Cover as shown
 - Cable cover bracket
 - (2) #4-40 x ¼" pan head screw

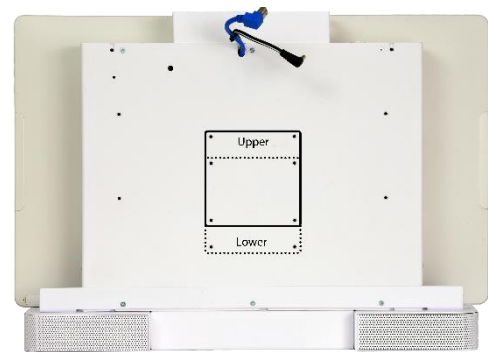


Figure 1 - Rear Mounting holes

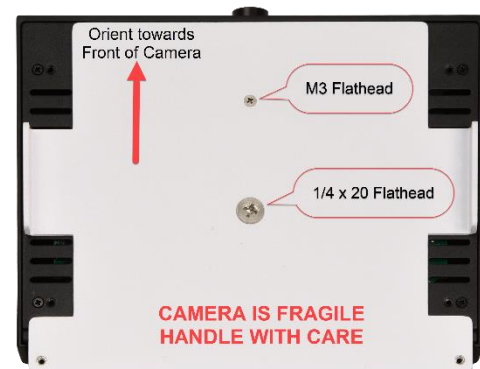


Figure 2 - Camera bracket attachment



Figure 3 - Camera Mounting

6. Attach the Speaker deflector bracket to the bottom of the CLINiC
 - a. Remove the three screws from the base of the CLINiC and reserve for re-use.
 - b. Orient the bracket as shown so the sound deflects to the front of the CLINiC.
 - c. Attach the deflector using the 3 screws previously removed.



Figure 2 - Remove screws from underside



Figure 1 - Attach sound deflector

Log into Admin

- Plug the network/ethernet cable into the ethernet port on the device
- Plug in power and allow the unit to boot. This may take a minute or two.
- The codec will be assigned an IP address from the DHCP server.
- To locate the IP address, swipe up from the bottom of the screen and then select More, then Settings. Write down or photo the serial number and IP address as shown.



- To log into the administrator Web interface for the device, your computer will need to be on the same network as the CLINiC.
- Enter the IP address in any browser on your PC, you should reach the Login screen for the CLINiC. When queried, enter the user Name: admin and the last 6 digits of the serial number as the password.
- See the [Poly Administration Guide](#) for additional features of the codec.